


# Flowflex™

## SARS-CoV-2

### Antigen Rapid Test (Self-Testing)



A rapid test for the detection of SARS-CoV-2 antigens in nasal specimens. For self-testing use.

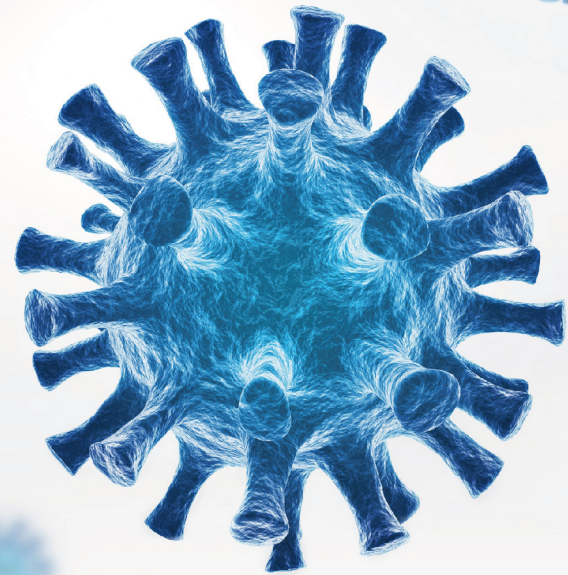

The Global Leader of Rapid Test with **26** Years Experience.



Fast



Accurate



Easy to Use



Reliable

**ACON®**

# Flowflex SARS-CoV-2 Antigen Rapid Test (Self-Testing)

The SARS-CoV-2 Antigen Rapid Test is a lateral flow test for the qualitative detection of the antigen from SARS-CoV-2 in nasal swab specimens directly from individuals suspected of COVID-19 within the first seven days of the onset of symptoms. The test can also test specimens from individuals without symptoms.

- **Specimen:** Nasal swab specimens
- **Test Time:** Results at 15 min
- **Shelf life:** 24 months
- **Storage temperature:** 2-30°C
- **Accuracy:** 98.8%
- **Sensitivity:** 97.1%
- **Specificity:** 99.5%


## Test Procedure

### 1. Test Preparation

A. Open your test kit:



B. You should have:

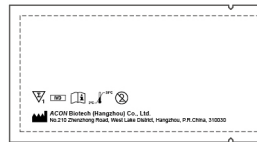


Test cassette



Swab



Extraction Buffer Tube

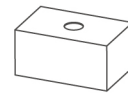

Tube Holder



Insert

### 2. Specimen Collection

COLLECTION BY AN ADULT CAREGIVER




OR

SELF COLLECTION




1

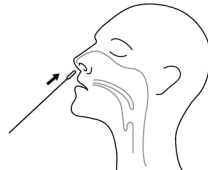


Insert the tip of the swab into one nostril. Using gentle rotation, push the swab less than 2.5 cm from the edge of the nostril.

2

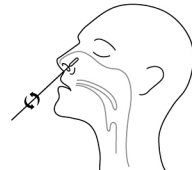


Rotate the swab 5 times brushing against the inside of the nostril.

3

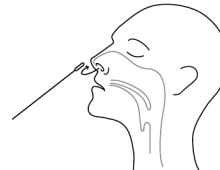




Remove the swab and insert it into the other nostril.

4

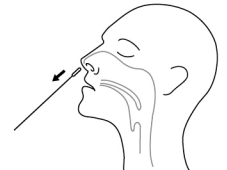


Remove swab from the nostril.


### 3. Sample Preparation

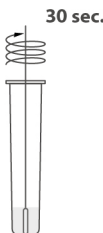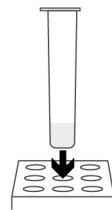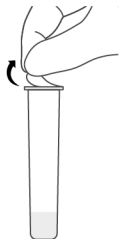


30 sec.

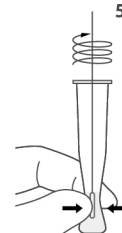

5x

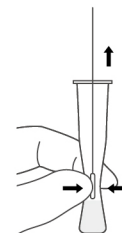

### 4. Running the Test

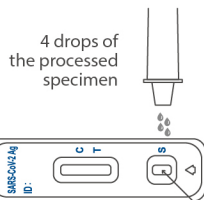

4 drops of the processed specimen

Sample Well



15-30 min.



Negative



Positive



Positive



Invalid



Invalid

## Ordering Information

Product Name	Catalog No.	Format	Specimen	Package
Flowflex SARS-CoV-2 Antigen Rapid Test (Self-Testing)	L031-11855	Cassette	Nasal swabs	1 Test/Kit 20 Kits/Box
	L031-118A5	Cassette	Nasal swabs	5 Tests/Kit



ACON Biotech (Hangzhou) Co., Ltd.  
No.210 Zhenzhong Road, West Lake District, Hangzhou, P.R.China, 310030  
Tel: +86-571-8118 9706 / Fax: +86-571-8777 5781 / Email: information@aconlab.com.cn  
[www.aconbio.com](http://www.aconbio.com)